


Additional Supplies:

**RAZOR**  
Shaving excess arm hair will improve chances of a successful collection.


## 1. Prepare



**1.** Wash hands before use. Healthcare workers must wear gloves during collection.

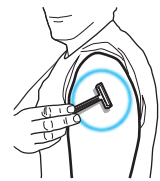

**2.** Device will be placed on upper arm - for best results remove any excess hair.

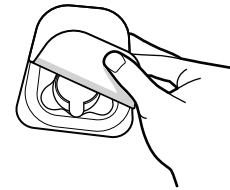


**3.** Collect all needed items. Remove device from tray.



**4.** Twist off cap from compatible tube.



**5.** Press tube into device until snug with fill lines facing out.

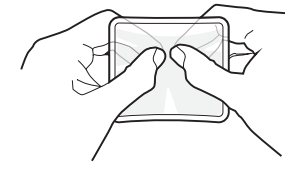


**6.** Follow heat pack instructions to activate. If pack is not available, use other warming method such as hand.



**Increase blood flow at collection site**



**7.** Rub heat pack or hand up and down shoulder for 2 minutes to warm and increase blood flow.

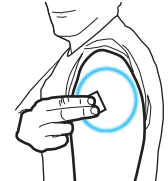

**8.** Clean area with alcohol pad and allow to dry.


## 2. Collect




**You may not feel the device. Leave on 5 minutes or less if full.**




**9.** Remove clear cover from red button.

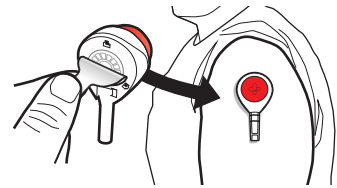


**10.** Peel tab away from device and stick device to upper arm. Hang arm straight down at side.

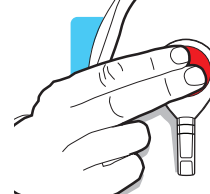

**11.** Press button all the way down ONCE and release. Set timer for 5 minutes. Watch tube fill using mirror as needed.

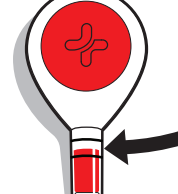


**12.** Blood may not appear for first 1-2 minutes. Remove device at 5 total minutes or sooner if blood reaches top line before 5 minutes.

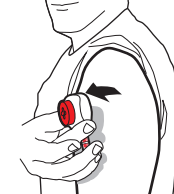

**13.** Remove device by peeling from one side. If needed, stop bleeding with bandage.



**14.** Remove tube from device with a slight twist and pull down.

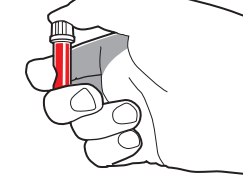

**15.** Snap cap fully onto tube.



**16.** Invert tube per manufacturer's labeling.

## 3. Disposal



**17.** Follow local guidelines for discarding used materials.



**18.** Wash hands after use.

## 4. Transport

**19.** If returning blood sample to a lab by parcel service, follow provided shipping instructions.

### INTENDED USE

The Tasso+ is a single-use blood lancing device intended for obtaining microliter capillary whole blood samples.

### DEVICE DESCRIPTION

Tasso+ is a sterile, disposable blood lancing device. When collecting a sample, the device is held onto the skin by a mild adhesive. A sterile lancet punctures the skin with a press of the button, simultaneously this creates a slight vacuum that helps capillary blood flow under gravity into a compatible tube. After a predefined time period, or when the tube is full, the device is peeled off the skin. The tube is removed and capped for transport. The lancet is single use and automatically retracts into the device in a safe position after use.

### MATERIALS PROVIDED

Tasso Blood Collection Device

### DISPOSAL

Follow local guidelines for discarding used sharps as a biohazard material.

### MATERIALS REQUIRED (NOT PROVIDED)

- Gloves (for Healthcare Providers)
- Heat Pack or other Arm Warming method
- Compatible Tube, refer to manufacturer's labeling
- Bandage
- Alcohol Swab

Max Length: 5 mm  
Max Depth: 2 mm



First time using Tasso?  
Watch our instructional video:  
[www.tassoinc.com/tassoplus-video](http://www.tassoinc.com/tassoplus-video)  
or scan the QR code to access.



Contact us for help: 1-800-257-2370

PN0389-01, 06

### WARNINGS

Not intended for more than one use. Do not use on more than one patient. Improper use of blood lancets can increase the risk of inadvertent transmission of bloodborne pathogens, particularly in settings where multiple patients are tested.

### PRECAUTIONS

- For use only on a single patient. Discard the entire device after use.
- Do not use after expiration date.
- Single use only.
- Keep out of reach of children.
- Minor bruising or residual marks may occur at the sample collection site.
- Do not resterilize.
- For external use only.
- Use while seated as fainting may occur with any blood sampling procedure.
- Not for use on infant heels.
- Prescription use only.
- Refer to manufacturer's instructions for individual components.
- Do not use if device packaging has been opened or damaged.
- Users must validate use of product for their specific diagnostic assay.
- Use personal protective equipment when handling biohazardous materials.

### SYMBOLS

REF	Catalog number	Single use only
LOT	Lot number	Do not resterilize
	Do not use if package is damaged	Store at 18-28°C
	Use by date	Transport at 25-85% RH
	Consult instructions for use	Rx ONLY Prescription use only
	This device contains sharps	STERILE R Sterilized by irradiation



Tasso, Inc.  
400 2nd Ave W.  
Seattle, WA 98119 USA  
+1 800-257-2370 | [www.tassoinc.com](http://www.tassoinc.com)

Tasso



**Tasso+**  
Instructions For Use